- EN WellO<sub>2</sub> – USER MANUAL
- SV WellO<sub>2</sub> – BRUKANVISNING
- NO WellO<sub>2</sub> – BRUKERHÅNDBOK
- FI WellO<sub>2</sub> – KÄYTTÖOHJEKIRJA
- ES WellO<sub>2</sub> – MANUAL DEL USUARIO
- FR WellO<sub>2</sub> – MANUEL D’UTILISATION
- DE WellO<sub>2</sub> – GEBRAUCHSANWEISUNG
- RU WellO<sub>2</sub> – РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ

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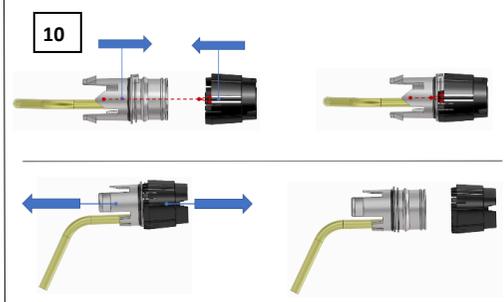
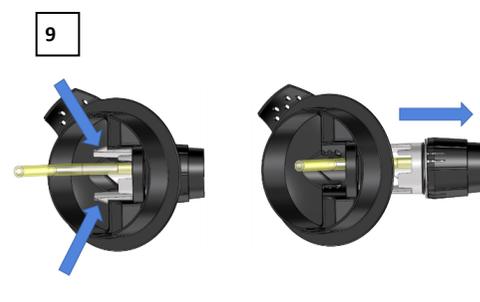
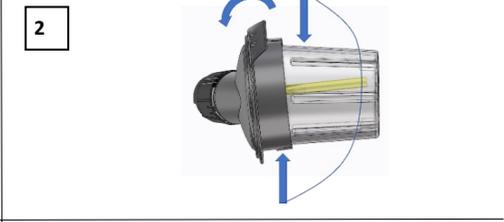
## WellO2 — USER MANUAL

WellO2 is a breathing exercise device. It opens, moisturises and soothes airways, and strengthens respiratory muscles. The WellO2 innovation combines inspiratory / expiratory muscle training and steam inhalation. It is intended for home use.

WellO2 is useful for everyone with breathing difficulties. Also, the athletes, the professional voice users, and those suffering from shallow breathing or snoring, benefit from the device.

WellO2 provides a natural way to improve breathing. Resisted inhaling and exhaling strengthens respiratory muscles, opens airways and helps the inhaled steam to penetrate deep into the airways. The steam moisturises and clears mucus from the airways, and alleviates airway irritation.

WellO2 has been designed to be used at home. It is easy to use, and its hygiene can be maintained with the simple in-built program and with the personal hygienic parts.

Adjustable breathing resistance and temperature makes the device versatile. You may be a poor and shallow breather or professional athlete; the device offers you the challenge that fits to your current condition.

Breathing exercise with the WellO2 makes breathing easier. Its regular use intensifies inhaling and exhaling power, helps to adopt deep breathing technique, and protects airways against stressful effects of poor air quality.

### CONTENTS

<b>Page</b>	<b>Chapter</b>
6	1. Introduction
6	2. Safety information
6	2.1 General warnings
7	2.2. General cautions
7	2.3 Dry-boil protection
8	2.4 Protection from Electromagnetic radiation
8	3. Disposal
8	4. Contraindications
8	5. Introduction for using the device
9	6. Using the device
10	7. Instructions and useful tips for breathing exercise
11	8. Cleaning the device
12	9. Execution of the cleaning program
13	10. Descaling
14	11. Troubleshooting
15	12. Specifications
15	13. Warranty

## 1. Introduction

Please read the instructions for use carefully before using the device. If you are uncertain about any part of the instructions, we recommend that you contact your retailer or the manufacturer (info@wello2.com) for more information.

## 2. Safety Information

This chapter provides important information on safe and efficient use. Before using the product, you must read and understand the following safety-related information. Keep the instructions available while using the product. The manufacturer accepts no responsibility for inappropriate use.

Warnings and cautions are given in several parts of these instructions. You must always pay attention to them whenever they are given.

⚠ Warnings indicate a potential personal injury if the instructions are not followed.

! Cautions indicate a potential risk of damage to the product and/or property if the instructions are not followed.

### 2.1 General warnings

⚠ If any part of the device has been damaged or it is not working properly, do not use or dismantle it. Contact your retailer or the manufacturer for further advice.

⚠ Any maintenance operations except cleaning and descaling as advised in these instructions must be performed by the manufacturer's representative.

⚠ Plug the device into an grounded mains socket only.

⚠ Check that the voltage indicated on the rating plate of the device corresponds to that of the mains.

⚠ Do not use any power base other than the one provided by the manufacturer.

⚠ Always place the device on a dry, flat, and stable surface. Keep the device and the power cord away from external heat sources, wet or slippery surfaces, or sharp edges.

⚠ Never turn the device on without filling the water vessel as instructed.

⚠ Immediately unplug the device from the mains if not operating correctly.

⚠ Do not immerse the device, the power base (15), the power cord or the plug (14) in water or any other liquid.

⚠ **Do not wash the main body (8) or power base (15) in dishwasher.**

⚠ When filling the device, avoid spillage on the connectors, the power cord or the plug (14).

⚠ Do not use the device when your hands are wet or your feet are bare.

⚠ Do not open the lid when the cleaning program is running.

⚠ Do not put your hands over the ventilation hole (11) during any operation.

⚠ Ensure that the lid (9) is properly locked when using the device.

⚠ Never open the lid (9) while the water is hot, especially immediately after the cleaning process.

⚠ Always use the handle (13) when moving the device.

- ⚠ Never move the device while it is heating water. Let the device cool down for at least 20 minutes after the cleaning process is completed before moving it.
- ⚠ Unplug the power base (15) as soon as you have finished using the device and always before cleaning it. Leave it unplugged when the device is not in use.
- ⚠ Do not unplug the device from the mains by pulling the power cord (14). It is recommended that you wind the power cord back into the power base after you have finished using the device.
- ⚠ Do not replace or remove any parts of the device when it is connected to power base (15) and mains.
- ⚠ Always store the device dry. Never leave water inside the device or tubing.
- ⚠ WellO2 should not be used by persons (including children) with reduced physical, sensory or mental capabilities, unless under supervision by a person responsible for their safety.
- ⚠ Children under 12 years are not allowed to use the device unless supervised by adult. Keep the device and its cord out of reach of children aged under 12 years. Do not allow children to play with the device or any of its parts.
- ⚠ Always remain vigilant when the device is on. Do not perform any other activities when using the device.
- ⚠ Hot surfaces, hot water and steam may cause burns and scalds. If accident occurs, immediately move the injured person away from the heat source and cool the injured area with cold water or ice for 10 to 30 minutes. Contact a doctor if necessary.

## 2.2 General cautions

- ! This device is designed for indoor use only.
- ! Only use for breathing exercise. Do not use as an electric kettle. Do not insert any external objects inside the device.
- ! Use only for a maximum of 30 minutes at a time. Let the device cool down for 30 min before turning it on again.
- ⚠ Any modification of the device is not allowed since it may hamper safety. Product warranty do not apply if the device is modified anyhow.
- ! Use only the descaling methods described in chapter **10 DESCALING**.
- ! Do not use scorers to clean the device.
- ! Protect the device from humidity, direct sunlight and frost during storage.
- ! To avoid possible adverse effects of humidity and condensation, let the device reach the room temperature if it has been in cold.

## 2.3 Dry-boil protection

Never use the device without appropriate amount of water in it. This device switches off automatically if you switch it on with too little or no water in it. In this case, let the device cool down for 10 minutes and then lift it off the power base. Fill it with clean and cold water so that the water level is in between the MIN and MAX fill level marks (7). The device is ready for use again.

## 2.4 Protection from electromagnetic radiation

This device complies with Electromagnetic Compatibility (EMC) standards of European Union.

## 3. Disposal

Dispose the product according to your national regulations for disposal of electronic devices.

## 4. Contraindications

Do not use Wello2 without consulting your physician in case you have or suspect to have any of the following conditions.

- Emphysema
- Pneumothorax
- Unstable cardiovascular status
- Recent myocardial infarction or pulmonary embolus
- Chest, abdominal, or cerebral aneurysms
- Recent eye surgery (including cataract)
- Recent surgery of thorax or abdomen
- Epilepsy
- Nose bleeding
- Pregnancy

## 5. Introduction for using the device

Remove the device from its packaging and check that all the following parts are included:

- main body (8) and lid (9) with the basket (10) attached
- power base (15) with power cord (14)
- 3 mouthpieces (5), (each in a sealed plastic bag)
- breathing regulator (3), (in a sealed plastic bag)
- extension hose (6), (in a sealed plastic bag)
- nasal mask (16), (in a sealed plastic bag)

1. Check that all outer and inner packages are intact and remove the main body, the power base and the accessories from their plastic bags.
2. Remove the lid (9) from the device (8) as shown in **FIGURE 1** and remove the basket (10) from the lid (9) as shown in **FIGURE 2**.
3. Disassemble the breathing regulator (3) by pulling the two parts off each other as shown in **FIGURE 10**.
4. Rinse water vessel of the device and the detached parts with clean water.
5. Execute the cleaning program according to the chapter **9 EXECUTION OF THE CLEANING PROGRAM** starting from step 1.
6. Add clean drinking water into main body's (8) vessel so that the water level is in between the MIN and MAX fill level marks.
7. Attach the connecting element (2) and resistance setting ring (4) together as shown in **FIGURE 10**.
8. Attach the breathing regulator (3) to the lid (9) as shown in **FIGURE 3**. Check that the tube is at the lower site of the regulator as indicated in the **FIGURE 3**.

9. Attach the basket (10) to the lid (9) taking care that the tubing is placed in the correct position as shown in **FIGURE 4**.

10. Put the lid (9) in place and close it by turning it 90 degrees clockwise as shown in **FIGURE 5**.

⚠ If any of the parts are clearly damaged or missing from the package do not use the device!

⚠ The breathing regulator (3) contains two check valves. Do not attempt to remove the valves from their housing!

⚠ When using the device for the first time or if you are not sure that the device is clean, run the cleaning program according to chapter **9 EXECUTION OF THE CLEANING PROGRAM** before using it.

⚠ Do not use water that looks, smells or tastes suspicious.

⚠ Use clean drinking water only. If you are not sure about the quality of your tap water, use bottled still water.

⚠ Never fill the water vessel above the MAX fill level mark! Overfilling may lead to malfunction or / and scalding injuries.

⚠ Always use your personal mouthpiece (5) to avoid microbial cross-contamination.

## 6. Using the device

1. Fully unwind the power cable (14) located in the power base (15) and place the power base on a dry, flat and stable surface such as a table.

2. Place the main body (8) on the power base (15) and plug the power cable (14) into the grounded mains supply.

3. WellO2 has three pre-set temperature buttons (12) marked with one (●), two (●●), and three (●●●) dots (see **FIGURE 7**). Select the desired temperature by pressing one of the pre-set buttons until the device emits a short sound signal and the indicator light of the selected button blinks slowly.

4. When the water has reached the selected temperature, the device emits a long sound signal and the heating process stops.

5. The indicator light for the selected temperature stops blinking and stays on.

6. Lift the main body (8) off the power base (15) and place the main body on a dry, flat and stable surface. Be careful with the hot lid (9).

7. Attach the mouthpiece (5) to the breathing regulator (3) as in **FIGURE 6**. For better breathing posture you can attach the extension hose (6) to the mouthpiece (5). You may also use the nasal mask (16).

8. Adjust the exhaling/inhaling resistance by turning the resistance setting ring (4) on the breathing regulator as shown in **FIGURE 8**. We suggest beginning with the lowest resistance and gradually increase it as your breathing power develops.

9. After use, clean the device as described in chapter **8 CLEANING THE DEVICE**.

⚠ **IMPORTANT!** Make sure not to select the cleaning program as this will take the water to boil. Inhaling hot steam can be very unpleasant.

⚠ **NEVER** perform the breathing exercise while the device is on the power base.

⚠ Always make sure that the breathing regulator (3) is operating correctly. It includes two check valves regulating the direction of the air flow. Blowing into the device should generate bubbles in the water vessel whereas inhaling should bring only steam into your mouth, not water. If the water still rises to the regulator (3), stop using the device and check the part assembly. Replace the breathing regulator (3) as a new one if necessary.

⚠ Beware the hot steam that comes out of the ventilation hole (11) during the exercise.

! Please note that the air flowing through the device during the breathing exercise may cause whistling sound.

## 7. Instructions and tips for starting a breathing exercise

### The very first exercises

It is recommended to start training with the lowest vapor temperature (●) and breathing resistance (position 0).

Breathing exercise is performed by repeating exhalations and inhalations, either continuously or as a single exhaled and inhaled breath. While exercising, you may take breaks in between ex- and inhalations if you feel it is too strenuous or you need to cough. You may also experience fast mucus loosening which is a normal phenomenon if you have accumulated mucus in the airways. In this case, the mucus is just coughed, after which you may continue using the device normally.

### Breathing exercise without nasal mask

Select the breathing resistance from the breathing regulator, and the temperature from the operating buttons. Begin with the lowest breathing resistance (position 0) and the lowest steam temperature (●).

The device resists your breathing in both exhaling and inhaling directions. You may use higher breathing resistance later when you have become familiar with the device and the exercise.

You may select from the 3 pre-set temperatures:

(●) app. 45 °C

(●●) app. 55 °C

(●●●) app. 65 °C

Select the temperature according to your own feeling.

#### ***Expiratory muscle training***

1. *Fill up your lungs with air and exhale evenly into the device via mouthpiece or extension hose for app. 10-15 seconds until your lungs are almost empty.*
2. *Breathe freely a few times*
3. *Repeat points 1 and 2 for 1-4 times according to your feelings.*

ATTENTION: This may feel very hard in the beginning but as soon as you learn the correct exhaling technique and get used to the device, the exercise gets easier. You may also lower the resistance by exhaling slower.

#### ***Inspiratory muscle training***

1. *Empty your lungs freely and immediately fill them up by inhaling evenly from the device via mouthpiece or extension hose for 10-15 seconds.*
2. *Hold your breath for 1-3 seconds, after which breathe freely a few times.*
3. *Repeat the points 1 and 2 for 1-4 times according to your feelings.*

As soon as you learn the technique and your breathing power increases, you may repeat ex- and inhalations continuously without breathing freely in between them. You may make 5-15 reps at a time.

The strain of the exercise can be controlled either with a higher breathing resistance (0-3) or by inhaling and exhaling into the device slower or faster. In this case, the back pressure generated by the device decreases or increases and the exercise gets easier or harder, respectively. It is always important to listen to your own body and exercise according to your own performance and capability.

The device generates steam from hot water by blasting bubbles. With a small breathing resistance and high blowing power, the device generates more steam than with high resistance and low blowing power. By increasing breathing resistance, also the exhaling and inhalation times get usually longer. Exhaling and inhalation are always carried out with a constant flow.

If you use the device for extensive training of inspiratory and expiratory muscles, you may add power by:

- increasing the breathing resistance from the breathing regulator
- increase air flow by exhaling and inhaling faster
- making more exhaling and inhalation reps

For steam inhalation only, hold the resistance in the 0 position. Note that the heat produced by the appliance decreases during the exercise.

## Breathing exercise with the nasal mask

The WellO<sub>2</sub> package includes a separate nasal mask designed to deliver hot steam into the upper airways. Always use the lowest breathing resistance (0) when training with the WellO<sub>2</sub> nasal mask. Keep in mind that the nose senses the heat delicately. Always start with the lowest steam temperature (●).

1. Blow into the device through the extension hose (6) without a nasal mask to achieve steam.
2. Attach the mask (16) to the end of the hose (6) and press it tightly over the nose. Inhale the steam through the mask until your lungs are full.
3. Breathe freely a few times.
4. Repeat steps 1 and 2 according to your own feelings for 1-4 times.

Once you are familiar with the nasal mask and your breathing power increases, you may add reps gradually up to 10-15. The nasal mask exercise can also be done by exhaling into the nasal mask and then inhaling from the nasal mask.

Breathing through the nasal mask causes small vibrations, which can improve mucus clearance from the airways.

## 8. Cleaning the device

1. Let the device cool down for 20 minutes.
2. Detach the mouthpiece (5) from the breathing regulator (3).
3. Open and remove the lid (9) by turning it 90 degrees counter clockwise and then lift it up as shown in **FIGURE 1**.
4. Drain the water completely. Be careful as the water may still be hot.
5. Detach the basket (10) from the lid (9).
6. Detach the breathing regulator (3) from the lid (9) by gently pressing the two snaps while pulling the parts apart as shown in **FIGURE 9**.
7. Separate the two parts on the breathing regulator by pulling them apart as shown in **FIGURE 10**.
8. Rinse the water vessel with plenty of clean water and let dry.
9. Rinse the separate components with clean water, drain the excess of water and let them dry.
10. Clean the device every time you use it.
11. Periodically clean the device as described in chapter **9 EXECUTION OF THE CLEANING PROGRAM**.

⚠ **Never immerse the device (8), the power base (15), or the power cord and the electric plug (14) in water since it may cause risk of electric shock.**

⚠ **Never wash the main body (8) or power base (15) in dishwasher.**

⚠ Do not use scouring pads. Do not use strong or abrasive cleaning agents.

⚠ The breathing regulator (3) contains two check valves. Do not attempt to remove them.

⚠ Always store the device dry and upright position. Never leave water inside the device or tubing.

## 9. Execution of the cleaning program

Before running the cleaning program, carefully clean the device as described in chapter **8 CLEANING THE DEVICE**.

1. *Attach the connecting element (2) with the tubing to the lid (9) so that the two snaps are firmly in place as shown in **FIGURE 3**.*
2. *Place the resistance setting ring (4) and the mouthpiece (5) into the basket (10)*
3. *Attach the basket (10) to the lid (9) as shown in **FIGURE 4**.*
4. *Add clean and cold water so that the water level is in between the MIN and MAX fill level marks (7).*
5. *Close the lid (9) as shown in **FIGURE 5**.*
6. *Place the device on the power base (15) and plug the power cable (14) into the grounded mains supply.*
7. *Select the cleaning program by pressing the cleaning button  until the device emits a long sound signal.*
8. *The indicator light on the cleaning button blinks quickly and the device starts heating the water. Do not open the lid while water is heating.*
9. *When the water reaches boiling point the heating process stops but the cleaning program continues for approx. 20 minutes, after which:*
  - *The device emits a long sound signal.*
  - *The indicator light on the cleaning button stops blinking and remains steady. Be careful with the hot surfaces of the main body (8).*
10. *When the main body (8) has cooled down, lift it off the power base (15), open the lid (9) and remove the basket (10) carefully.*
11. *Drain the excess water from the connecting element (2), resistance setting ring (4), and mouthpiece (5) and leave them dry.*
12. *Pour the water out of the device and wipe it dry.*

⚠ Do not open the lid (9) while the cleaning program is on.

⚠ To prevent microbial growth, it is important to periodically run the cleaning program.

⚠ It is recommended to run the cleaning program:

- *before using the device for the first time*
- *after every fifth usage but minimum after every tenth usage*
- *after a prolonged period of non-use (e.g. one month)*
- *whenever the device has been used by others*

⚠ Always store the device dry. Never leave water inside the device or tubing.

⚠ The device emits steam from the ventilation hole (11) during the cleaning process. Do not put your hands over the ventilation hole (11) during any operation.

⚠ Never fill the water vessel above the MAX fill level mark! Overfilling may lead to malfunction or / and scalding injuries.

! Always fill the device with cold water as the cleaning program can be started only when the temperature of the water is below 40 °C.

## 10. Descaling

Soluble salts in the water may hamper the correct operation of the device due to the formation of lime scale in the water vessel, valves and tubing system.

Detach the removable parts as described in chapter 8 Cleaning the device.

Place the detached components into the basket (10) and attach the basket to the lid (9) as shown in Figure 4.

### Descaling options:

#### A) Vinegar:

- fill the device with 0.5 L of vinegar
- close the lid (9) as shown in **FIGURE 5** and leave to stand for 1 hour **without heating**
- drain the device
- carefully rinse the device and all parts 5 – 6 times

#### B) Citric acid:

- fill the device with 0.5 L of water
- add 25 g of citric acid
- close the lid as shown in **FIGURE 5** and leave to stand for 15 min **without heating**
- drain the device
- carefully rinse the device and all parts 5 – 6 times

#### C) descaling products designed for plastic kettles:

- follow the instructions supplied with the descaling product to prepare 0.5 L of descaling solution and pour it into the device
- close the lid (9) as shown in **FIGURE 5** and leave to stand according to the instructions supplied with the descaling product **without heating**
- drain the device
- carefully rinse the device and all parts 5 – 6 times

Repeat the descaling procedure if necessary.

! Descale regularly, preferably at three months intervals, or more frequently if the water used contains soluble salts.

## 11. Troubleshooting

This chapter summarises the most common problems that you may encounter with the product.

- If the device emits three short sound signals when one of the pre-set buttons is pressed and any indicator light blinks quickly:
  - *The temperature of the water is higher than the selected temperature. Let the device cool down for a few minutes before putting it on again.*
  - *Accumulation of scale may cause incorrect reading of water temperature and trigger the alarm. Descale according to the procedure described in chapter **10 DESCALING** if necessary.*
- If the device emits three short sound signals when the cleaning button is pressed and the indicator light on the cleaning button blinks quickly:
  - *The temperature of the water is higher than 40 °C. Let the device cool down or replace the water with clean and cold water.*
  - *Accumulation of scale may cause incorrect reading of water temperature and trigger the alarm. Descale according to the procedure described in chapter **10 DESCALING** if necessary.*
- If the device does not start heating the water:
  - *The device is equipped with a Dry-boil protection. Please refer to chapter **2 SAFETY INFORMATION** (dry boil protection) for details.*
  - *The device is not correctly positioned on the power base. Remove it from the base and position it correctly.*
- If the water temperature seems to be much higher or lower than expected.
  - *Accumulation of scale may cause an incorrect water temperature reading. Descale according to the procedure described in chapter **10 DESCALING**.*
- The inhaled vapour tastes or smells of plastic
  - *Plastic odours may evaporate when the device is used for the first few times. If the problem persists, fill the device with water to the MIN fill level mark and add two teaspoons of sodium bicarbonate (baking soda). Run the cleaning program and drain the water. Rinse the device 2- 3 times.*

△ Do not use the device if it has been dropped, it leaks or you notice any visible damage in the power cord, the plug or the power base.

△ Do not use the device when the power cord (14) is damaged.

## 12. Specifications

Manufacturer	Hapella Oy, Maaherrankatu 9, 70100 Kuopio, Finland
Input Voltage	230 VAC, 50 Hz
Input Power	1000 W
Dimensions (H x W x D)	240 mm x 240 mm x 150 mm (9.4" x 9.4" x 5.9") incl. the power base
Weight	1055 gr (37 oz.) including the power base
Operating temperature	+10 to +30 °C (+50 to +86 °F)
Storage temperature	-20 to +50 °C (-4 to +122 °F)
Operating humidity	20% to 90% non-condensing
Storage humidity	20% to 90% non-condensing

## 13. Warranty

Wello2 device has product warranty of two (2) years excluding the replaceable parts (below):

- *silicon tube (1)*
- *connecting element (2)*
- *breathing element (3)*
- *resistance setting ring (4)*
- *mouthpiece (5)*
- *extension hose (6)*
- *nasal mask (16)*

The product has no right of return if the seal on the outer package is broken.

Further information is available at [www.wello2.com](http://www.wello2.com) or [info@wello2.com](mailto:info@wello2.com).

Specifications are subject to change without notice.

The Wello2 brand is the property of Hapella Oy.

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